



Slim MED™ ADVANCED is a medical device certified in accordance with the provisions of Directive 93/42/EEC.

Please carefully read the enclosed instructions for use.

### PURPOSE

Slim MED™ ADVANCED is a lipid binder to be used in the treatment and prevention of overweight in order to prevent diseases related to excess of weight and body fats. This medical device is intended to help with:

- treatment of excess of weight,
- weight control,

by means of lowering fat intake from diet and therefore reducing the caloric intake for restoring a healthy energetic balance and supporting a weight management program.

### MODE OF ADMINISTRATION AND COMPOSITION

Package with 60 capsules

- Main ingredient: 1 capsule of Slim MED™ ADVANCED contains 500 mg of KiOnutrime-CsG® (biopolymer).
- Other components:

Magnesium stearate, Silica; capsule: Hypromellose (HPMC), Titanium dioxide.

### USE

Slim MED™ ADVANCED is intended for overweight and obese adults (Body Mass Index  $\geq 25$ ) wishing to lose or control weight in the context of a weight management program.

Recommended intake to support:

- Weight reduction: 2 capsules three times a day
- Weight control: 1 capsule three times a day

To be taken 15 minutes before a main meal with a large glass of liquid and within the scope of a balanced and varied diet.

Suitable for long term use, Slim MED™ ADVANCED is recommended to be taken for 12 weeks.

### ADDITIONAL INFORMATION

No flavour enhancer, preservative.

No lactose, iodine, gluten or cholesterol.

Slim MED™ ADVANCED is produced with natural ingredients and does not contain raw materials of animal origin. Slight variations in colour and taste have no impact on the product's effectiveness.

### SIDE EFFECTS

A temporary feeling of satiety can take place after taking Slim MED™ ADVANCED. This feeling normally disappears after a short time.

Persons taking Slim MED™ ADVANCED may occasionally experience gastro-intestinal disorders such as constipation, dyspepsia, stomach aches, uncomfortable bloating, abdominal discomfort, nausea, vomiting or diarrhoea.

Rare: hyper sensibility to one or more ingredients.

In the event of persistent side effects, subjects should discontinue the treatment and should talk to a healthcare professional.

### CONTRAINDICATIONS

Do not use in case of allergy or intolerance to one of the compounds present in the formulation.

Do not take Slim MED™ ADVANCED if you have inflammatory diseases of the gastrointestinal tract (e.g. acid reflux disease, gastritis, ventricular or duodenal ulcers, Crohn's disease, ulcerative colitis, diverticulitis), signs of intestinal obstruction (impending or existing mechanical or paralytic ileus), gastroparesis in diabetics with neuropathy, intestinal polyps and existing severe digestive disorders.

Do not use if you have a

medical history of constipation.

Do not use if underweight (Body Mass Index (BMI)  $<18.5$ ).

Not suitable for babies, infants or children.

Adolescents should use Slim MED™ ADVANCED only after consulting a doctor.

### PREGNANCY AND BREAST-FEEDING

Do not use during pregnancy and breastfeeding in order not to jeopardise the supply of important nutrients to the embryo or baby.

### PRECAUTIONS FOR USE

As an insoluble gel develops in the stomach, it is therefore

necessary to ensure a sufficient water intake. In principle you should drink at least two litres of water or unsweetened liquids per day.

Do not exceed the recommended daily dose. Taking Slim MED™ ADVANCED at higher doses than recommended does not improve efficacy of the product, but can promote potential side effects.

The high fat binding capacity of Slim MED™ ADVANCED might impact absorption of active ingredients of lipidic nature from medicinal products. This concerns e.g. the hormones in contraceptive pills or in preparations used in the

treatment of menopause disorders and vitamin products. In order not to jeopardise the efficacy of such medications, at least four hours should pass between their intake and that of Slim MED™ ADVANCED. In order to avoid any interaction with other substances, subjects should ask systematically their doctor or pharmacist about concomitant treatments.

If required, in order not to fall below the daily requirement of fat soluble vitamins A, D, E and K, a multivitamin product can help cover the vitamin requirement.

Slim MED™ ADVANCED is without carbohydrate unit incidence. However, people

suffering from diabetes should use Slim MED™ ADVANCED only after consulting a doctor.

Do not chew or bite through the capsules. Capsules must not dissolve in the mouth.

### CONDITIONS OF STORAGE

Store in a dry place at room temperature

Keep out of reach of children.

Verify the blister integrity before use.

Do not exceed the deadline of use being provided on the packaging.

Medical Device



### MANUFACTURER

KitoZyme SA  
Rue de Milmort, 680  
BE 4040 Herstal Belgium

[www.kitozyme.com](http://www.kitozyme.com)

Information accurate as of:  
2016-02